



Meril

Delivering the ideal flow for accurate anastomosis

NAUTICATM

Intracoronary Shunt





NAUTICA™


Intracoronary Shunt

The NAUTICA™ Intracoronary Shunt is specially designed to provide a bloodless operative field during vessel anastomosis procedure. The simple yet effective design temporarily allows the blood flow towards the distal by-pass route from anastomotic site in the vessel. The NAUTICA™ Intracoronary Shunt facilitates anastomosis during Coronary Artery Bypass Grafting and Beating heart surgery.


The NAUTICA™ Intracoronary Shunt is a radio-opaque one piece device consisting of a flexible silicone hollow shaft having bulb shaped tips. The Radiopaque Tag, indicates the diameter of the bulb and is attached to the shunt via suture. The NAUTICA™ Intracoronary Shunt is safe and easy for insertion and is removed from the vessel just prior to the completion of anastomosis.

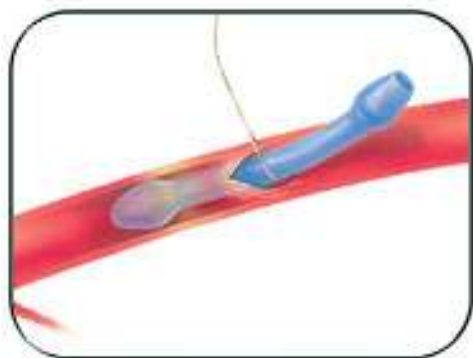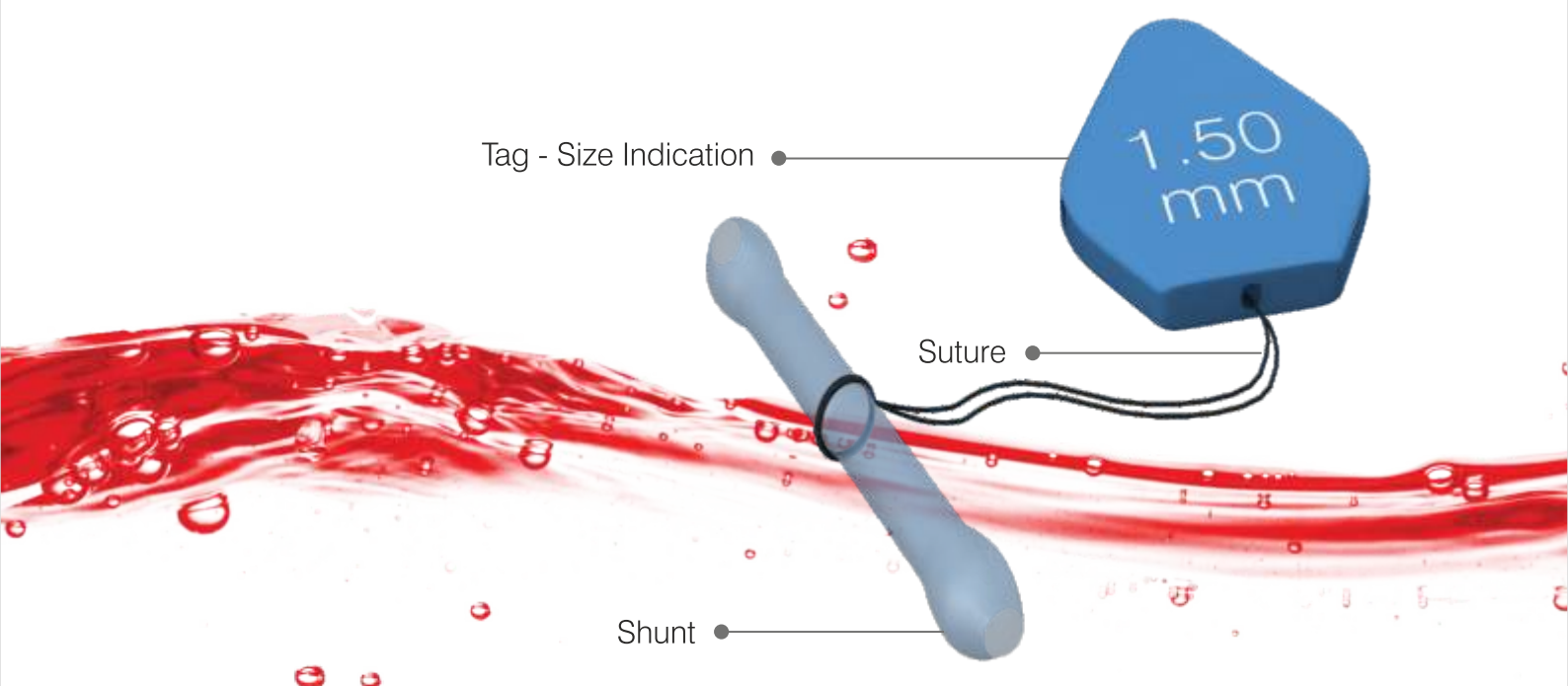

Easy Insertion

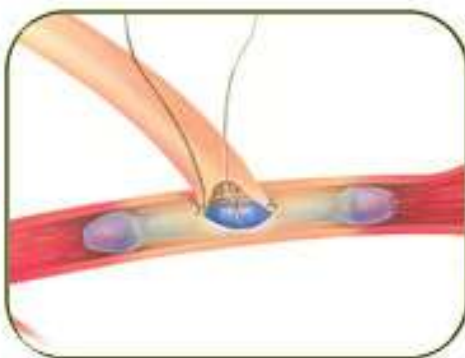

Vessel Occlusion, Stenting and
Distal Perfusion

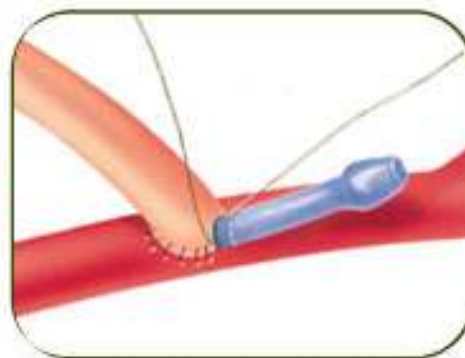


Easy Atraumatic Removal

CLINICAL BENEFITS

Available in various sizes from 1.25mm - 3.00mm facilitating an ideal fit.

Soft silicone bulb tips allows easy insertion and atraumatic removal through the insertion site.

Broad bulb tip ends secures with the arterial wall and builds a blood less operative field.

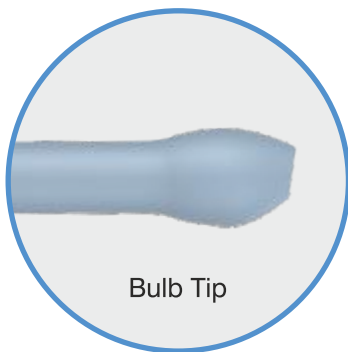


Bulb ø (mm)

1.25	1.50
1.75	2.00
2.50	3.00

Distinctively designed for better visualization at the site of anastomosis.

Moulded into a single flexible silicone piece potentially eliminates the risk of intima injuries.

Uniform lumen diameter facilitates greater blood flow for better performance.

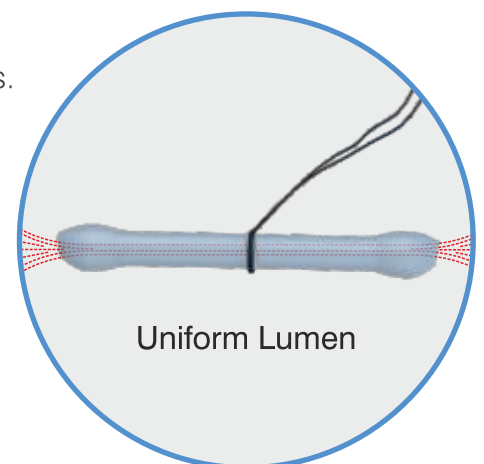

Ordering Information

SR. NO	NAUTICA INTRACORONARY SHUNT	PRODUCT CODE
1	NAUTICA Intracoronary Shunt 1.25 mm	ICS125
2	NAUTICA Intracoronary Shunt 1.50 mm	ICS150
3	NAUTICA Intracoronary Shunt 1.75 mm	ICS175
4	NAUTICA Intracoronary Shunt 2.00 mm	ICS200
5	NAUTICA Intracoronary Shunt 2.50 mm	ICS250
6	NAUTICA Intracoronary Shunt 3.00 mm	ICS300

NAUTICA™ Intracoronary Shunt is a single-use only and cannot be resterilized.

Contact our Sales representative or Meril Life Sciences for ordering information, in-service support materials and technical specifications.



Meril Life Sciences India Pvt. Ltd.

M1-M2, Meril Park, Survey No 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396 191, Gujarat, India.
T : +91 260 3052 100 F : +91 260 3052 125 E : askinfo@merillife.com W : www.merillife.com

Global Offices: Meril Turkey | Meril South America | Meril Germany